

URGENT PRODUCT RECALL

July 13, 2026

Dear Valued McKesson Customer:

Solventum has notified McKesson Medical-Surgical Inc. (MMS) of an Urgent Product Recall Notice regarding specific lot(s) of their 3M Comply Lead Free Steam Indicator Tape. This notice has been issued because these lots may contain rolls without indicator ink present due to a manufacturing printing nonconformance. Affected product first shipped December 10, 2025.

McKesson Medical-Surgical Inc. has taken appropriate action per this notice.

For questions regarding this notification, please contact Solventum at **(800) 228-3957**.

A review of our records indicates that your company may have purchased items included in this notification. Carefully review the information in this letter and follow the instructions provided below.

Refer to the table below for a list of affected item(s) and lot number(s) distributed by McKesson Medical-Surgical

MMS #	MFG Catalog #	Description	Affected Lot(s)	Exp. Date
775746	1322-18MM	TAPE, INDICATOR LEAD FREE STEAM 0.70"X60YDS (28RLS/CS)3M	251106	5/6/2027
775747	1322-24MM	TAPE, INDICATOR LEAD FREE STEAM 0.94"X60YDS (20RLS/CS)3M	251106	5/6/2027
1174305	1322-24MM	TAPE, INDICATOR STEAM 0.94"X60YDS (L-SEQUESTER) (20RLS/CS)	251106	5/6/2027

McKesson Customer Instructions:

- 1.) Immediately discontinue use of any product matching the affected item(s) and lot number(s) listed above. If you have no product matching the affected item(s) and lot number(s), no further action is needed
- 2.) A copy of the Urgent Product Recall from Solventum has been included for reference.
- 3.) If you have product affected by this notice, fill out the McKesson Reply Form and return it to our Corporate Customer Service Center via email at MMSRecalls@McKesson.com or fax at **(866) 871-0270**. To ensure timely credit to your account and support the completion of this notice, please respond within 30 days. **Please note:** Credit will only be issued for the affected Lot(s) of the product(s) listed above and entered on the reply form. **Please place a new order for replacement product if there is an immediate need.**
- 4.) Please dispose of any affected product in your possession in accordance with your institution's policies and procedures.
- 5.) If you have further distributed any of the item(s) referenced in this notification, provide your accounts with a copy of this notification.

We sincerely apologize for any inconvenience this notice may have caused you and your staff. If you have any questions about information provided in this communication, please contact our **McKesson Medical-Surgical Recall Message Center** at MMSRecalls@McKesson.com or call **(800) 688-8840**.

Thank you for your prompt attention,
McKesson Medical-Surgical Inc.

McKesson Medical-Surgical Inc.

www.mckesson.com
RC-2026-150

McKesson Medical-Surgical Inc.
 Product Recall Reply Form: RC-2026-150
 Solventum - 3M Comply Lead Free Steam Indicator Tape

July 13, 2026

Complete this reply form and return all pages immediately via email to MMSRecalls@McKesson.com or fax at **(866) 871-0270** should you have affected product.

To ensure timely credit to your account and support the completion of this notice, please respond within 30 days.

Date: _____ Ship to Acct Number: _____

Your Name: _____ Email Address: _____

Phone Number: _____ Fax Number: _____

Account Name: _____

Address: _____

City, State Zip: _____

☐ I acknowledge that I DO HAVE product affected by this notification and have disposed of this product in accordance with my institution's policies and procedures.

Qty	Unit of Measure	Affected Lot #(s)	MMS #	MFG Catalog #	Description
			775746	1322-18MM	TAPE, INDICATOR LEAD FREE STEAM 0.70"X60YDS (28RLS/CS)3M
			775747	1322-24MM	TAPE, INDICATOR LEAD FREE STEAM 0.94"X60YDS (20RLS/CS)3M
			1174305	1322-24MM	TAPE, INDICATOR STEAM 0.94"X60YDS (L-SEQUESTER) (20RLS/CS)

***Discard Affected lot numbers only**

*** The affected lot number(s) are listed on the McKesson customer letter. Please dispose of affected product in accordance with your institution's policies and procedures.**

Credit will only be issued for product(s) listed above with affected lots.

If you have any questions about information provided in this communication, please contact the McKesson Recall Message Center at MMSRecalls@McKesson.com or call (800) 688-8840.

June 30, 2026

Subject: URGENT Voluntary Product Recall - 3M™ Comply™ Lead Free Steam Indicator Tape 1322-18MM and 1322-24MM Lot 251106

Dear Distributor:

As a valued Solventum (formerly 3M Health Care) customer, we wanted to make you aware of our recent decision to conduct a voluntary recall of one (1) lot of the 3M™ Comply™ Lead Free Steam Indicator Tape 1322-18MM and 1322-24MM distributed between December 10, 2025 through June 4, 2026. Refer to specific product affected in the table below.

SAP ID	Legacy ID	Cat. No.	Lot Number	Production Date	Expiration Date
7000138229	CT-0609-3465-1	1322-18MM	251106	06 Nov 2025	06 May 2027
7000138230	CT-0609-3466-9	1322-24MM	251106	06 Nov 2025	06 May 2027
7100276701	70-2011-9058-7	1322-24MM	251106	06 Nov 2025	06 May 2027

These lots may contain rolls without indicator ink present due to a manufacturing printing nonconformance. If the indicator tape roll does not contain indicator ink, the product will not provide visual distinction between processed and unprocessed units. The volume of potentially impacted rolls within the identified lots are low. Although this is not a safety concern and there are no reported injuries associated with this issue, the product is being recalled in an abundance of caution and as part of our commitment to ensuring the highest levels of quality.

Actions Requested:


1. Immediately cease distribution of the affected lot of 3M™ Comply™ Lead Free Steam Indicator Tape 1322-18MM and 1322-24MM.
2. Notify all customers who have purchased the affected product using the **Customer Letter** and **Customer Acknowledgement Form** provided.
3. Determine if any of the affected products are in inventory.
4. Complete the **Distributor Acknowledgement Form** with the following:
 - Acknowledge that you have read and understood this letter ~~whether or not you have any affected product in stock at your facility;~~ and
 - Indicate if you have any product in stock and dispose of this material per facility policy. ~~Note, replacement product may be requested from Solventum for any affected product within inventory that has been disposed of.~~
 - Email the completed form and replacement request to SolventumFieldSafetyAction@montage72.com

This notification only applies to the 3M™ Comply™ Lead Free Steam Indicator Tape 1322-18MM, 1322-24MM lot 251106 and does not apply to any other 3M™ Indicator Tapes.

If you have additional questions, contact the Solventum (formerly 3M Health Care) Customer Helpline at 1-800-228-3957 option 6.

We apologize for any inconvenience this may cause you. Thank you for your cooperation.

Sincerely,

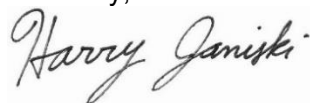

Harry Janiski
Vice President, Global Quality Assurance
Solventum