
1. The air compressor should be kept at least 6-8 inches away from the wall for ventilation and to keep the compressor from overheating.
2. The compressor should be set to at least 20 psi then lock the dial by pushing in so the settings cannot be adjusted.
3. The air compressor should be wiped down weekly to avoid dust build up.
4. The air inlet filter in the back of the compressor should be rinsed out weekly. Let it air dry before placing back into the compressor.
5. The air inlet filter should be changed when it turns from gray to brown.

## **AEROSOL COMPRESSOR**

Oxygen adaptor can be placed either between the jet nebulizer and corrugated tubing or between the corrugated tubing and the trach mask.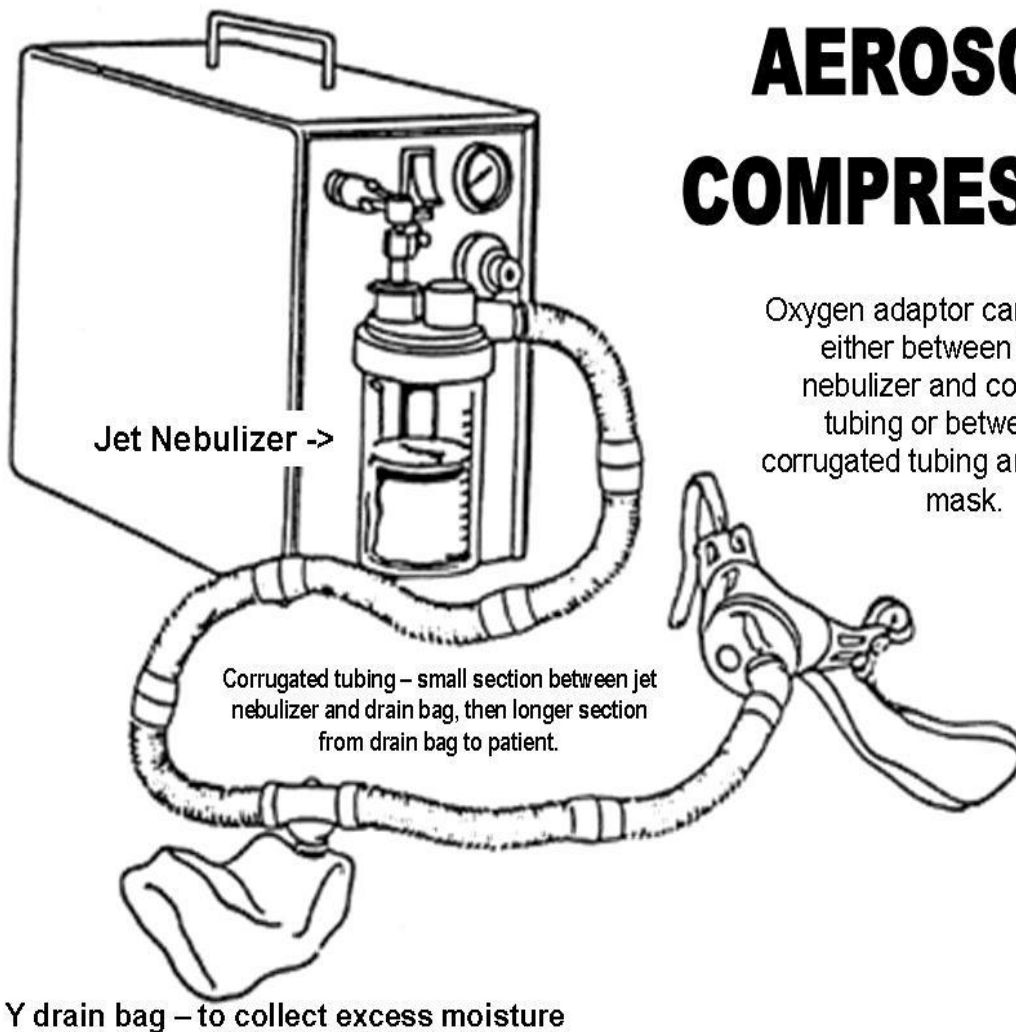

Call your local CareLinc for supplies, questions and concerns.

# DEVILBISS®

## *DeVilbiss*® Heavy-Duty Compressor Instruction Guide Model 8650D

**WARNING**—Unit not approved for use with a ventilator.

**CAUTION**—Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

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Made in U.S.A.

MEDICAL EQUIPMENT  
AIR COMPRESSOR  
40HN

WITH RESPECT TO ELECTRICAL SHOCK, FIRE AND MECHANICAL  
HAZARDS ONLY IN ACCORDANCE WITH UL60601-1

**TABLE OF CONTENTS**

Important Safeguards . . . . .	3
Important Parts of Your 8650D Compressor . . . . .	3
How to Operate Your 8650D Compressor . . . . .	4
Compressor Cleaning . . . . .	5
Filter Cleaning/Changing . . . . .	6
Maintenance . . . . .	6
Specifications . . . . .	7
Warranty . . . . .	7

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## IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

**▲ DANGER** Urgent safety information for hazards that will cause serious injury or death.

**▲ WARNING** Important safety information for hazards that might cause serious injury.

**CAUTION** Information for preventing damage to the product.

**NOTE** Information to which you should pay special attention.

## SAVE THESE INSTRUCTIONS

### INTENDED USE

The DeVilbiss 8650D is used with a pneumatic nebulizer to administer humidified air or medication into the airways to treat respiratory disorders. The device is intended for home health care and institutional use. Users of this device are adult and pediatric patients with tracheostomies or suffer from asthma, cystic fibrosis, and chronic obstructive pulmonary disease.

**NOTE**—The DeVilbiss 8650D is to be used only when prescribed by a physician and operated in accordance with this guide.

### IMPORTANT PARTS OF YOUR 8650D COMPRESSOR

#### Figure A

1. Pressure gauge
2. Power switch
3. Pressure adjustment knob
4. Air outlet
5. Extension tube (optional 8650D-613)
6. Tubing connector (optional 7025D-613)

#### Figure B

1. Fan filter
2. Power connector/Fuse holder

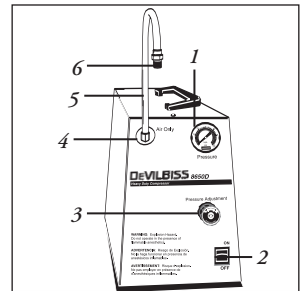
Figure A

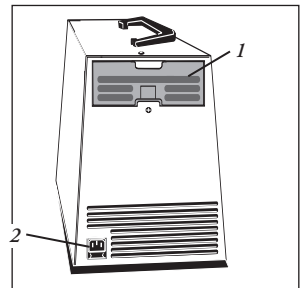
Figure B

## HOW TO OPERATE YOUR 8650D COMPRESSOR

1. Make sure the power cord is unplugged from wall-outlet power source.

### **⚠ WARNING**

**Explosion Hazard. Do not operate in the presence of flammable anesthetics.**

2. Refer to *Important Parts of Your 8650D Compressor* for part identification.
3. Place the 8650D compressor on a flat surface in a position allowing for adequate air flow from the rear of the device.
4. For disposable nebulizer use, thread the optional tubing connector to either the compressor air outlet or optional extension tube, then attach one end of nebulizer tubing **Figure 1** or **Figure 2**.

**NOTE**—Follow specific instructions provided by physician for set up of both nebulizer and medication.

**CAUTION**—During high-humidity weather conditions, condensation water build-up can occur in the internal motor hose of the compressor. It is recommended that the 8650D be turned on and run for two minutes before the nebulizer/tubing is connected to the compressor.

5. For high output humidification, thread the bottle adapter directly to the extension tube air outlet **Figure 3**.
6. Before plugging in the power cord, make sure the power switch is in the "Off" position by pressing on the bottom half of the switch **Figure 4**. Plug the power cord into a properly grounded wall-outlet power source.

### **⚠ WARNING**

**Ground reliability can only be achieved when the power cord is connected to an equivalent receptacle marked "Hospital Grade."**

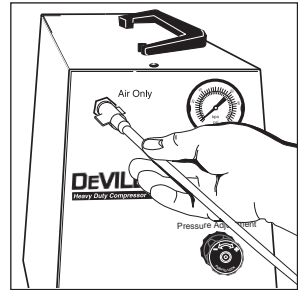
Figure 1

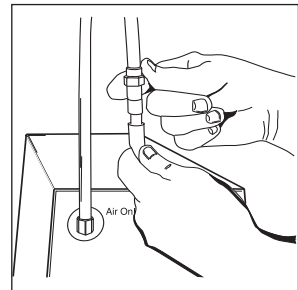
Figure 2

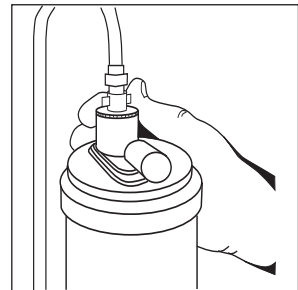
Figure 3

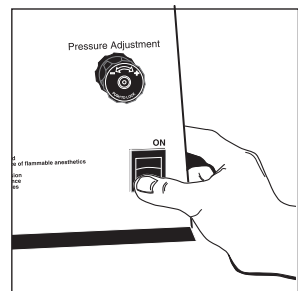
Figure 4

**CAUTION**—This model is designed to operate utilizing 115V, 60Hz. Do not attempt to operate this compressor on any other power source, as damage to the motor may result.

7. To start the compressor, press the power switch "On."

**NOTE**—Cold temperatures may cause a delayed motor start-up. Should this occur, place the compressor in a warmer room until the unit reaches room temperature. Then press the power switch again.

Compressor pressure may be increased by turning the pressure adjustment knob in a clockwise direction (knob must be pulled out to unlock) **Figure 5**. Pressure is decreased by turning the pressure adjustment knob in a counterclockwise direction. After adjusting the pressure as specified by your physician or the medication instructions, push adjustment knob to lock.

**NOTE**—For continuous use, do not exceed 50 psi maximum pressure.

**NOTE**—In order for unit to register a correct pressure gauge reading, nebulizer/tubing must be connected to the compressor.

### COMPRESSOR CLEANING

Be sure the power has been disconnected. Wipe the compressor cabinet with a clean cloth and any commercial disinfectant.

**CAUTION**—Do not submerge in water, as compressor damage will result.

**CAUTION**—Do not gas sterilize.

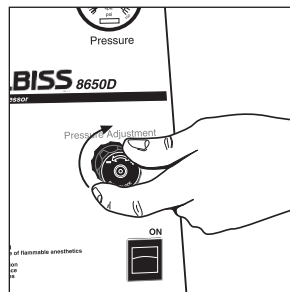
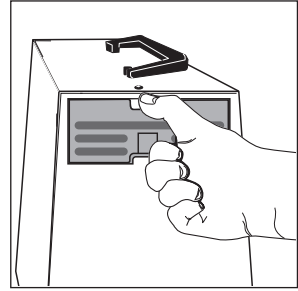
Figure 5

**FILTER CLEANING/CHANGING**

Your DeVilbiss 8650D compressor contains two filters:

**Fan Filter**—The fan filter refines the air taken into the unit to cool the compressor. It is located on the back of the compressor cabinet **Figure 6** and should be changed when deteriorated. During continuous use, it is recommended that the filter be cleaned weekly with warm soapy water, rinsed thoroughly, and dried completely.

*Figure 6*

**Air-Inlet Filter**—This filter is located inside the cabinet and should be changed every 12 months or more often as indicated by environmental conditions. It should be serviced by a Sunrise Medical provider's personnel or authorized service center.

**FUSES**

**⚠ WARNING**

**Risk of fire. Replace fuse only as marked 6.3A 250V.**

**MAINTENANCE**

All maintenance must be performed by a Sunrise Medical provider or authorized service center.

**⚠ DANGER**

**Electric shock hazard. Do not remove cabinet. All disassembly and maintenance must be done by a Sunrise Medical provider.**

**NOTE**—The DeVilbiss 8650D compressor contains no user-serviceable parts. If you believe your compressor is not working properly, contact your authorized Sunrise Medical home medical equipment provider.

MEDICAL EQUIPMENT

AIR COMPRESSOR

40HN

WITH RESPECT TO ELECTRICAL SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH UL60601-1

Care should be taken when this product is used in the vicinity of other electronic equipment concerning possible electromagnetic interference between them. They may need to be moved apart.

**SPECIFICATIONS**

<b>Compressor size (base unit):</b>	8 W x 12.3 H x 13 D (20cm x 31cm x 33cm)
<b>Compressor Weight:</b>	25 lbs. (11.3 kg)
<b>Sound Level:</b>	51 dBA maximum
<b>Electrical Requirements:</b>	5.0 amps, 115 VAC, 60 Hz
<b>Power Consumption:</b>	345 watts maximum
<b>Operating Pressure Range:</b>	5 - 50 psi
<b>Warranty:</b>	1 year

**ONE-YEAR LIMITED WARRANTY ON COMPRESSOR PARTS**

DeVilbiss Model 8650D Compressor is warranted to be free from defective workmanship and material for period of one year from date of purchase. Any defective part(s) will be repaired or replaced at Sunrise Medical's option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow instructions. If repair is necessary, contact your authorized Sunrise Medical provider or Sunrise Medical Customer Service Department at 800-333-4000 (814-443-4881) for instructions.

**NOTE** —This warranty does not cover providing a loaner compressor, compensating for costs incurred in compressor rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

**FOR SERVICE CALL YOUR AUTHORIZED SUNRISE MEDICAL PROVIDER:**

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Serie # \_\_\_\_\_

# OXYGEN CALL LIST

Monday  Tuesday  Wednesday  Thursday  Friday

**Automatic Stop**  Call Weekly  Every 2 Weekly  Call 6 months  Call Annually

*(except MCR 3-5 yr customers, identify who called in the order and how much they have left)*

**Note:**

1. Customers must be aware of their delivery day. They should call before noon on the business before their delivery day. We should not have "will call us" customers! After the first month, homefill **and** night time customers **with** internal filter concentrators will be called and have a home PMI at least every 6 months. Night time customers **without internal filters** must have a home PMI annually.
2. When making oxygen calls, if a customer only has 1-2 empty tanks, see if they have enough full tanks to last them another week.
3. Check on all **repeat supplies** with every 6 month / annual check. I.e. neb kits and filters, cpap / bipap supplies, etc
4. **Confirm** address, insurance and physician information with every supply order.

Name: \_\_\_\_\_ Phone: \_\_\_\_\_

(always make calls while looking at and updating the customers information in the computer)

(circle) **OXL OXQ OXM OXE OXD OXC OXB**

(circle) Home-Fill Cylinder: **Continuous Conserver** or **Helios** or **Conserver**

Comments/Supplies: \_\_\_\_\_

Other Equipment: \_\_\_\_\_

(circle) Rental Air Compressor with internal filter or **Rental Enteral Pump**

Annual check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

Annual check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

**(use form ADFM024 for all oxygen set ups, all rental swaps, and rental equipment checks)**

**Concentrator Set up date \_\_\_\_\_ internal filter = every 6 months / no internal filter = annually**

Physicians order: LPM \_\_\_\_\_ Duration \_\_\_\_\_ (check most current prescription / CMN on on-base, see DTW1004.)

PMI check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

PMI check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

PMI check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

PMI check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

PMI check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

PMI check due \_\_\_\_\_ Form ADFM024 initiated \_\_\_\_/\_\_\_\_/\_\_\_\_ By: \_\_\_\_\_ PMI Completed date \_\_\_\_/\_\_\_\_/\_\_\_\_

Account #: \_\_\_\_\_